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English

Intended use

EPS800 is an external 25 A power source for protective earth resistance measurements in compliance with the IEC 60601-1 standard. It is only to be used in conjunction with the UNIMET® 800ST (e.g. article number B96028010).

Safety instructions

Only electrically skilled persons or persons who are instructed correspondingly are authorised to use the EPS800. It is essential to follow

- the relevant regulations applying to work on electrical installations and
- the enclosed sheet "Important safety instructions for Bender products".

Functional description

The standard DIN EN 60601-1 (VDE 0750-1) requires to carry out a PE resistance test with 25 A or with 1.5 times the rated current of the device, whichever is greater ($\pm 10\%$), from a power source of 50 or 60 Hz over a period of 5...10 seconds. Therefore, in particular manufacturers of medical devices test the PE conductor of their devices during type tests in the manner specified.

Connected to the UNIMET® 800ST, the EPS800 provides a regulated AC power source which supplies a sinusoidal current of 25 A ($\pm 10\%$). The measured resistance should be between $0\ \Omega$ and $0.3\ \Omega$. If the electrical resistance of the PE conductor connection of the DUT exceeds $0.3\ \Omega$, the current will decrease with increasing resistance.

For the 25 A PE resistance test an individual measurement number (#103) is to be used. This measurement number is only to be used for the single test "Protective earth resistance fixed cord 25 A (EPS800)" and for complete test sequences according to DIN EN 60601-1.

Device-specific safety information

Risk of destruction, if the device is exposed to severe shocks during the test.

Do not move the EPS800 during the test.

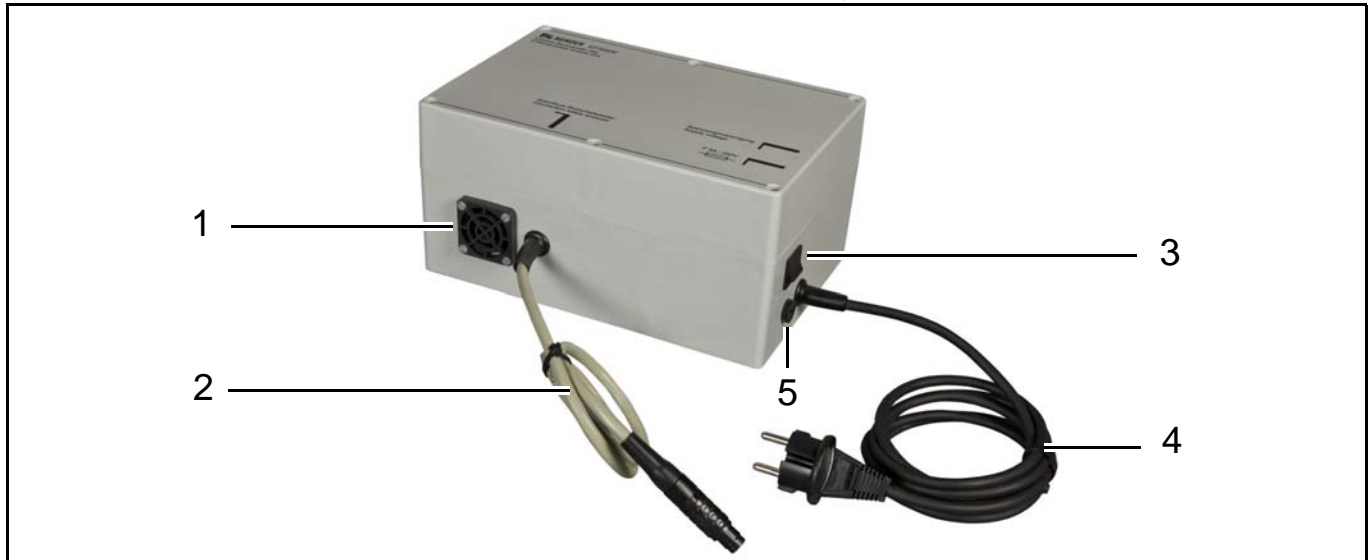
Early shut-down because the ventilation slots are not clear and obstructed.

The heat produced by the PE current is dissipated via the ventilation slots of the enclosure. A temperature monitoring unit eliminates the risk of overheating and switches the device off at approx. $70\ ^\circ\text{C}$. Always keep the ventilation slots clear and unobstructed!

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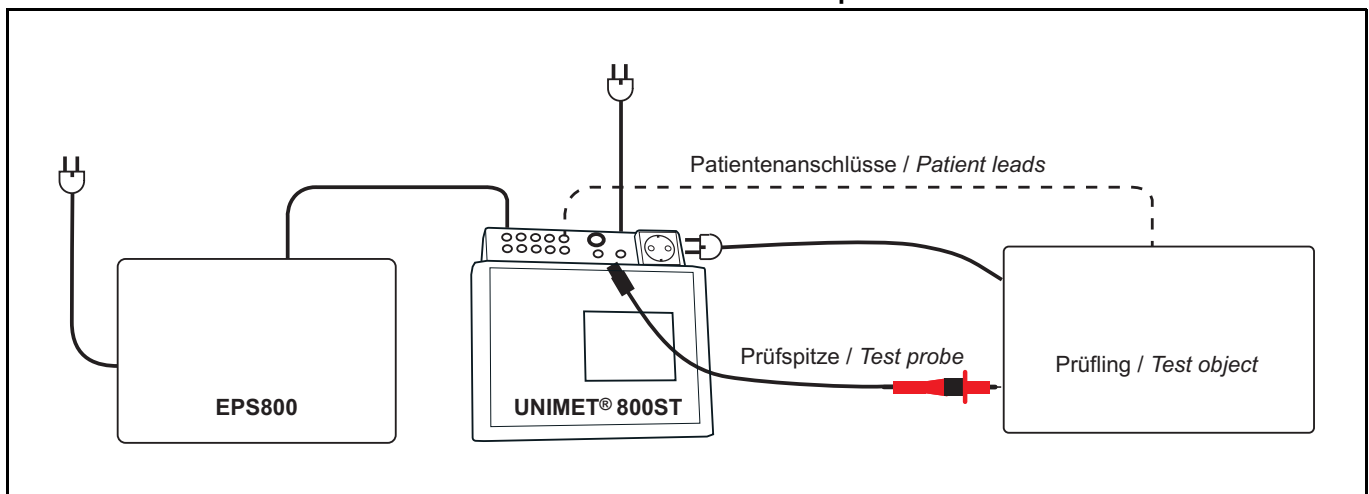
- 1 Ventilation slots
- 2 Connecting lead to UNIMET® 800ST
- 3 Power switch (The photo shows the power switch in ON position)
- 4 Permanently attached power cable for connection to the supply voltage
- 5 Micro-fuse (see "Technical data")

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Test setup

1. Plug the EPS800 connecting lead into the "EPS800" socket provided on the rear of the UNIMET® 800ST.
Note: The plug clicks into place and is secured against being pulled out accidentally. The plug can only be removed after pushing back the movable handle piece.
2. Connect the power cable of the EPS800 to the power outlet.
3. Connect the power cable of the UNIMET® 800ST to the power outlet.
4. Switch on the power switch of the UNIMET® 800ST.
5. Switch on the power switch of the EPS800. The sound of the internal ventilator can be heard.
6. Connect the DUT. Determine the test procedure by means of classification at UNIMET® 800ST.

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Safety shutdown in case of multiple consecutive PE resistance tests.

The high PE current produces heat. If the PE resistance test is repeated very often and without breaks during manual or semi-automatic test procedures or during a single test, the temperature monitoring integrated in the EPS800 will prevent overheating by switching the device off at approx. 70 °C. A message will be indicated on the UNIMET® 800ST display. After cooling down to 50 °C, a new PE resistance test can be carried out.

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Single test

Select measurement 0103 "Protective earth resistance fixed cord 25 A (EPS800)".

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Ordering details

Typ/Type	Art. No.
External power source 25 A for protective earth resistance measurements in accordance with IEC 60601-1 (only in conjunction with UNIMET® 800ST, Art. No. B 9602 8010), Plug/socket: D/D	B 9602 8050
External power source 25 A for protective earth resistance measurements in accordance with IEC 60601-1 (only in conjunction with UNIMET® 800ST, Art. No. B 9602 8014), Plug/socket: GB/GB	B 9602 8054
External power source 25 A for protective earth resistance measurements in accordance with IEC 60601-1 (only in conjunction with UNIMET® 800ST, Art. No. B 9602 8016), Plug/socket: CH/CH	B 9602 8056
External power source 25 A for protective earth resistance measurements in accordance with IEC 60601-1 (only in conjunction with UNIMET® 800ST, Art. No. B 9602 8017), Plug/socket: B/B	B 9602 8057
External power source 25 A for protective earth resistance measurements in accordance with IEC 60601-1 (only in conjunction with UNIMET® 800ST, Art. No. B 9602 8018), Plug/socket: US/US	B 9602 8058

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BENDER Group

All rights reserved.
Reprinting and duplicating
only with permission of the publisher.
Subject to change!
© Bender GmbH & Co. KG

D620013801

Bender GmbH & Co. KG
Londorfer Str. 65 • 35305 Grünberg • Germany
Postfach 1161 • 35301 Grünberg • Germany

Tel.: +49 6401 807-0
Fax: +49 6401 807-259

E-Mail: info@bender-de.com
Web: http://www.bender-de.com